

**From:** Smartheart@shl-telemedicine.com  
**Subject:** Smartheart Firmware Update  
**Date:** Mon, 11 Jul 2016 13:03:10 +0000

---

Dear Smartheart User,

We have sent a firmware update request to your device. [REDACTED]

Please follow the instructions below to complete the update:



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
Select **YES**

2. The update will begin to download.

You can track the progress by looking at orange bar.


This may take a few minutes.

**\*It is important not to turn off the device or exit the Smartheart app during this process.**



3. When the update is complete you will receive this message:

"Smartheart firmware version updated successfully."

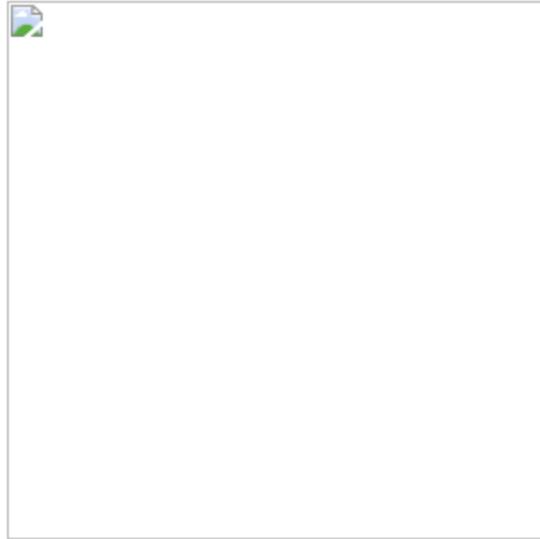

**The app will return to the "ECG Completed" screen and you can continue to use Smartheart as you did before.**

Kind Regards,

SHL Telemedicine  
 Smartheart

For more information visit:



*This message is for the named person's use only. It may contain confidential, proprietary or legally privileged information. No confidentiality or privilege is waived or lost by any mistransmission. If you receive this message in error, please immediately delete it and all copies of it from your system, destroy any hard copies of it and notify the sender. You must not, directly or indirectly, use, disclose, distribute, print, or copy any part of this message if you are not the intended recipient.*

#####  
#### Note: This message is for the named person's use only. It may contain confidential, proprietary or legally privileged information. No confidentiality or privilege is waived or lost by any mistransmission. If you receive this message in error, please immediately delete it and all copies of it from your system, destroy any hard copies of it and notify the sender. You must not, directly or indirectly, use, disclose, distribute, print, or copy any part of this message if you are not the intended recipient. SHL Telemedicine and any of its subsidiaries each reserve the right to monitor all e-mail communications through its networks. Any views expressed in this message are those of the individual sender, except where the message states otherwise and the sender is authorized to state them to be the views of any such entity. Thank You.

#####  
####